

DEVICE CONNECTION CABLE TYPE: I7

PHYSICAL CONNECTIONS

Ottobock Electric Wrist Rotator (10S17)

CONNECTION:
Coapt cable connection marked '**Close/Digital**' must plug into this side '2' of wrist

CONNECTION:
Coapt cable connection marked '**Open/Digital**' must plug into this side '2' of wrist

CONNECTION:
Coapt cable connection marked '**Rotate**' must plug into wrist port marked '3'

CONNECTION:
Power/battery cable (*not supplied by Coapt*) must plug into wrist port marked '1'

Motion Control Standard Wrist Rotator (5010045, 5010054, 5010055)

CONNECTION:
Coapt cable connection marked '**Rotate**' must plug into wrist port marked 'WRIST'

CONNECTION:
Coapt cable connection marked '**Open/Digital**' must plug into wrist port marked 'CH-A'

CONNECTION:
Power/battery cable (*not supplied by Coapt*) must plug into wrist port marked 'HAND'

CONNECTION:
Coapt cable connection marked '**Close/Digital**' must plug into wrist port marked 'CH-B'

SOFTWARE SETTINGS

WRIST SETTINGS:

No manual settings necessary for the Ottobock Standard Wrist Rotator or the Motion Control Standard Wrist Rotator

OTTOBOCK BEBIONIC DIGITAL SETTINGS:

For digital Ottobock bebionic hand operation via Coapt, the Ottobock bebionic hand should be kept in its default (out-of-the-box) settings state for proper operation.