

# DEVICE CONNECTION CABLE TYPE: K1

## PHYSICAL CONNECTIONS

### CONNECTION:


Coapt cable connections marked '1' through '6' must plug into the corresponding '1' through '6' of the MyoMaster (13E219)

### CONNECTION:

MyoMaster cable must plug into elbow port 'AXON' of DynamicArm Plus

### CONNECTION:

Coapt cable connection marked 'Connect to "1" or "2"' must plug into elbow port '1' or '2' of DynamicArm Plus

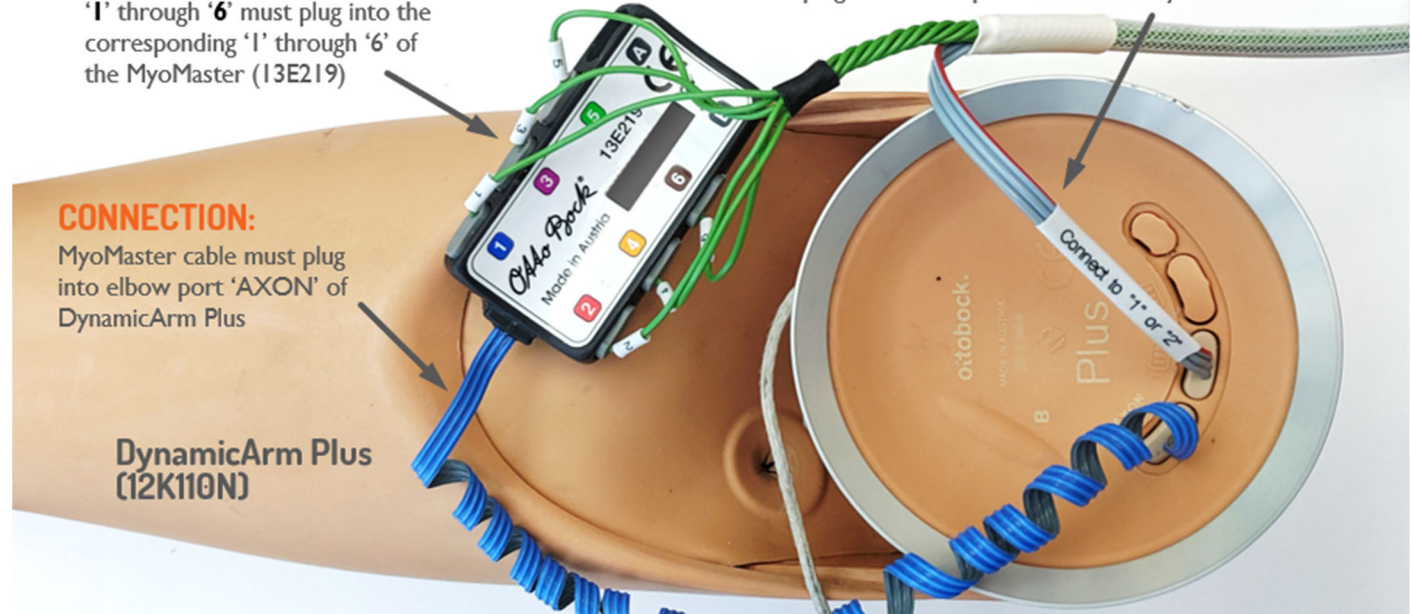

## SOFTWARE SETTINGS

### SETTINGS:

For proper operation of DynamicArm Plus 12K110N via Coapt, ensure all settings in the Ottobock ElbowSoft TMR 'Customizing' screen are set exactly as shown by clicking 'Default program' button.

