


DEVICE CONNECTION CABLE TYPE: K5

PHYSICAL CONNECTIONS

CONNECTION:

Coapt cable with 21-pin connector attaches firmly to mating connection on the elbow

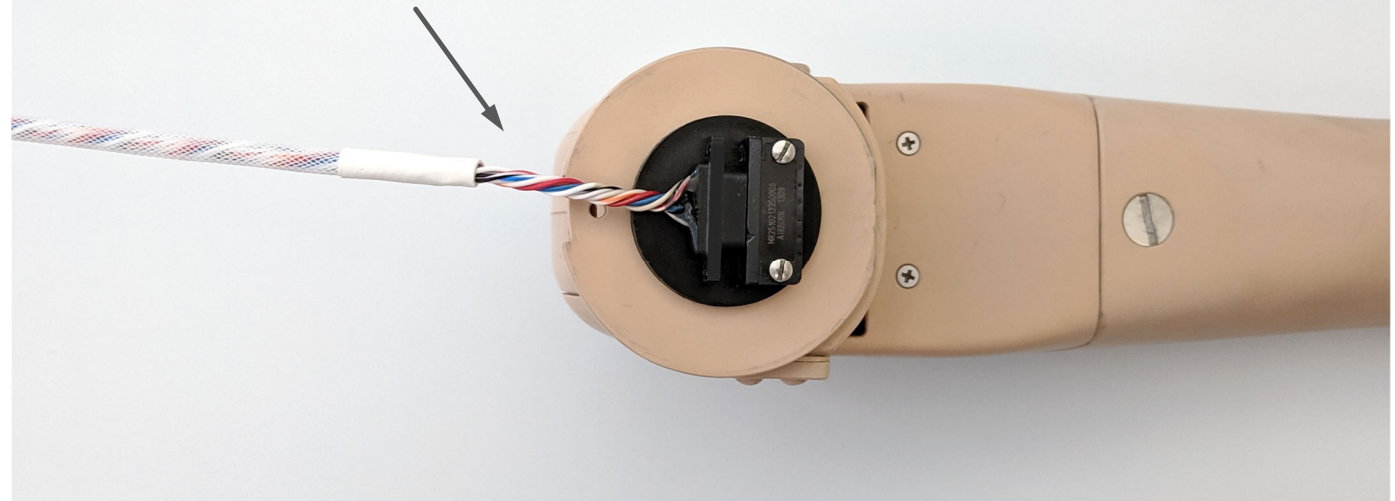

Motion Control Utah Arm 3+
with TASKA/bebionic Digital

SOFTWARE SETTINGS

MOTION CONTROL SETTINGS:

For proper operation via Coapt refer to settings shown in the on-screen guide available at “Connection Guide” in the Coapt Complete ControlRoom Gen2 user interface application.

TASKA HAND SETTINGS:

For digital TASKA hand operation via Coapt, TASKA settings will automatically be set upon each power-on

BEBIONIC HAND SETTINGS:

For digital bebionic hand operation via Coapt, refer to settings shown in the on-screen guide available at “Connection Guide” in the Coapt Complete ControlRoom Gen2 user interface application.