

DEVICE CONNECTION CABLE TYPE: K8

PHYSICAL CONNECTIONS

PRE-ASSEMBLED:

The Coapt controller (white PCB) will be pre-attached to the Espire unit. Do NOT attempt to remove it.


PRE-ASSEMBLED CONNECTION:

The Coapt COMPLETE CALIBRATE button is attached to the Coapt controller with its flex cable passing through the Espire Strain Relief Disc. Contact Coapt support for assistance detaching /attached this cable from the elbow.

PRE-ASSEMBLED CONNECTION:

Ensure the 2-pin, white Coapt aux connection is attached

College Park Espire Pro & Multifunction TD

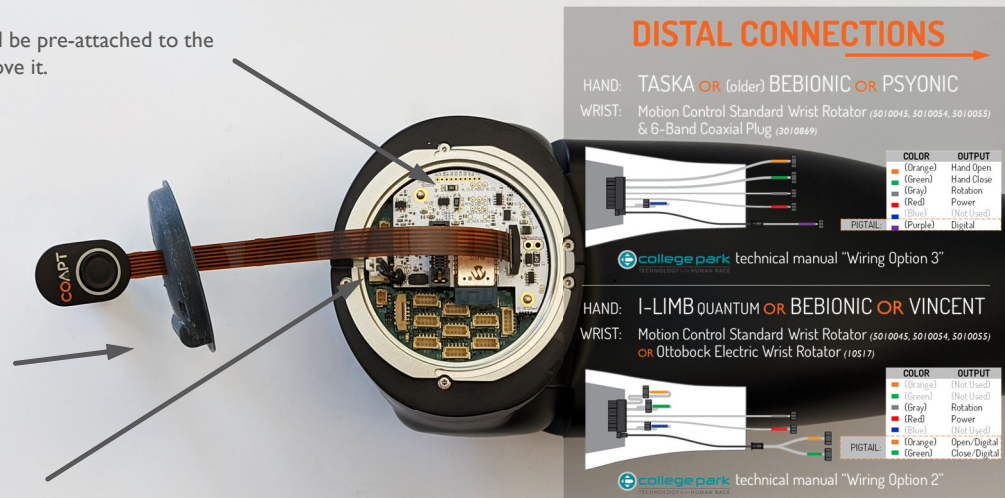

EMG CONNECTION:

The Coapt EMG Interface Cable (not shown here) for Espire has a special 90° exit. Ensure it is connected and secured once the Espire Strain Relief Disc is in place.

SOFTWARE SETTINGS

ESPIRE PRO SETTINGS:

In the Espire iOS user application, most screens will indicate the direct Coapt connection and not require specific settings.

I-LIMB SETTINGS:

For general i-Limb Digital operation via Coapt, most settings should be set to defaults in the i-Limb app. For proper operation of Gesture Control via Coapt, ensure 'CC' is selected as its trigger in the i-Limb app

TASKA HAND SETTINGS:

For digital TASKA hand operation via Coapt, TASKA settings will automatically be set upon each power-on

BEBIONIC HAND SETTINGS:

For digital bebionic hand operation via Coapt, refer to settings shown in the on-screen guide available at "Connection Guide" in the Coapt Complete ControlRoom Gen2 user interface application.